A QIAGEN / BD Company

**PAXgene<sup>®</sup>**  
***Blood RNA Tube***

EN	English	4-9
BG	Български	10-15
CS	Česky	16-21
DA	Dansk	22-27
DE	Deutsch	28-33
EL	Ελληνικά	34-39
ES	Español	40-45
ET	Eesti	46-51
FI	Suomi	52-57
FR	Français	58-63
HR	Hrvatski	64-69
HU	Magyar	70-75
IS	Íslenska	76-81
IT	Italiano	82-87
KK	Қазақша	88-93
LT	Lietuvių k.	94-99
LV	Latviski	100-105
NL	Nederlands	106-111
NO	Norsk	112-117
PL	Polski	118-123
PT	Português	124-129
RO	Română	130-135
RU	Русский	136-141
SK	Slovensky	142-147
SL	Slovenski jezik	148-153
SV	Svenska	154-159
TR	Türkçe	160-165
UK	Українська	166-171
ZH	简体中文	172-177
	Symbol and Mark Key	178-179

## EN - English

### I. Intended Use

The PAXgene Blood RNA System consists of a blood collection tube (PAXgene Blood RNA Tube) and nucleic acid purification kit (PAXgene Blood RNA Kit). It is intended for the collection, storage, and transport of blood and stabilization of intracellular RNA in a closed tube and subsequent isolation and purification of intracellular RNA from whole blood for RT-PCR used in molecular diagnostic testing.

**Performance characteristics for the PAXgene Blood RNA System have only been established with FOS and IL1B gene transcripts. The user is responsible for establishing appropriate PAXgene Blood RNA System performance characteristics for other target transcripts.**

### II. Summary and Explanation


The collection of whole blood is the first step in many molecular assays used to study intracellular RNA. A major challenge in this type of testing is the instability of intracellular RNA, which rapidly degrades within hours after blood collection. Furthermore, certain species of RNA, through the process of gene induction, increase in vitro after blood collection. Both in vitro RNA degradation and gene induction can lead to an under or overestimation of in vivo relative gene transcript number. The PAXgene Blood RNA Tube contains an additive that stabilizes the in vivo gene transcription profile by reducing in vitro RNA degradation and minimizing gene induction. When used in conjunction with the PAXgene Blood RNA Kit, the PAXgene Blood RNA Tube provides for accurate detection and quantitation of gene transcripts.

### III. Specimen Collection and Preparation for Analysis

#### A. Required Blood Collection Accessories (Not included with the PAXgene Blood RNA Tube)

1. Blood collection set such as the BD Vacutainer® Push Button Blood Collection Set or the BD Vacutainer Safety-Lok™ Blood Collection Set. *See Ordering Information.*
2. A BD Vacutainer Needle Holder must be used to ensure proper function. *See Ordering Information.*
3. A "Discard Tube" if the PAXgene Blood RNA Tube is the only tube being drawn. *See Ordering Information.*
4. Labels for positive donor identification of samples.
5. Alcohol swab for cleansing site.
6. Dry sterile gauze.
7. Tourniquet.
8. Needle disposal container for used needle or needle/holder combination.

#### B. Procedure for Specimen Collection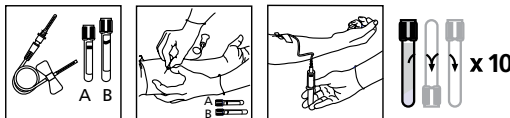

Draw Order:  
 Tube A - Discard Tube  
 Tube B - PAXgene Blood RNA Tube

1. Ensure that the PAXgene Blood RNA Tube is at 18–25°C prior to use and properly labeled with specimen identification.
2. If the PAXgene Blood RNA Tube is the only tube to be drawn, blood should be drawn into a "Discard Tube" prior to drawing blood into the PAXgene Blood RNA Tube so the interior volume of the blood collection set used during phlebotomy can be primed. Otherwise, the PAXgene Blood RNA Tube should be the last tube drawn in the phlebotomy procedure.

3. Using a blood collection set and a tube holder, collect blood into the PAXgene Blood RNA Tube using your institution's recommended procedure for standard venipuncture technique. **The following techniques shall be used to prevent possible backflow:**
  - a. Place donor's arm in a downward position.
  - b. Hold tube in a vertical position, below the donor's arm during blood collection.
  - c. Release tourniquet as soon as blood starts to flow into tube.
  - d. Make sure tube additives do not touch stopper or end of the needle during venipuncture.
4. **Allow at least 10 seconds for a complete blood draw to take place. Ensure that the blood has stopped flowing into the tube before removing the tube from the holder. The PAXgene Blood RNA Tube with its vacuum is designed to draw 2.5 ml of blood into the tube.**
5. Immediately after blood collection, gently invert the PAXgene Blood RNA Tube 8–10 times.
6. Store the PAXgene Blood RNA Tube upright at room temperature (18–25°C) for a minimum of 2 hours and a maximum of 72 hours before processing or transferring to refrigerator (2–8°C) or freezer (–20°C). See *Procedure for Freezing and Thawing Specimens in the PAXgene Blood RNA Tubes* for details if storage temperature of –70°C / –80°C is desired.

### C. Procedure for Freezing and Thawing Specimens Collected in the PAXgene Blood RNA Tubes

1. Stand the PAXgene Blood RNA Tube upright in a wire rack. **Do not freeze tubes upright in a Styrofoam™ tray as this may cause the tubes to crack.**
2. The PAXgene Blood RNA Tubes can be stored at –20°C and below. If tubes are to be kept at temperatures below –20°C, freeze them first at –20°C for 24 hours, then transfer them to –70°C or –80°C.
3. Thaw the PAXgene Blood RNA Tubes in a wire rack at ambient temperature (18–25°C) for approximately two hours. **Do not thaw the PAXgene Blood RNA Tubes at temperatures above 25°C.**
4. Carefully invert the thawed PAXgene Blood RNA Tubes 10 times.

**Note:** If the two-hour ambient temperature incubation was not carried out prior to freezing, incubate tubes at least another two hours after reaching room temperature before processing.

**Note:** The frozen PAXgene Blood RNA Tubes are subject to breakage upon impact. To reduce the risk of breakage during shipment, frozen tubes should be treated in the same manner as glass tubes. Users must validate their own freezing and shipping protocol for the PAXgene Blood RNA Tubes.

### D. Procedure for Specimen Preparation for Analysis

Blood samples must be processed according to the instructions provided with the PAXgene Blood RNA Kit. For catalog numbers, see *Ordering Information*.

## IV. Performance Characteristics

For a comprehensive listing of the performance characteristics for the PAXgene Blood RNA Tube, visit [www.PreAnalytiX.com](http://www.PreAnalytiX.com).

## V. Limitations

1. **Under-filling of the PAXgene Blood RNA Tubes will result in an incorrect blood-to-additive ratio and may lead to incorrect analytic results or poor product performance.**
2. The PAXgene Blood RNA System is not suitable for the collection and purification of viral RNA.
3. Blood samples obtained with the PAXgene Blood RNA Tube must be prepared only with the PAXgene Blood RNA Kit. For catalog numbers for the PAXgene Blood RNA Kit, see *Ordering Information*.
4. The quantity of blood drawn should be 2.5 ml per PAXgene Blood RNA Tube, but this volume might vary with altitude, ambient temperature, barometric pressure, tube age, venous pressure, and filling technique.
5. The PAXgene Blood RNA Kit is intended for purification of intracellular RNA from human whole blood with leukocytes counts between 4.8–11.0 × 10<sup>6</sup> leukocytes/ml.
6. Endotoxin not controlled. Blood and blood components collected and processed in the tube are not intended for infusion or introduction into the human body.


## VI. Warnings and Precautions

- Contents of this tube are irritating to skin.
  - After inhalation, supply fresh air: consult doctor in case of complaints.
  - After skin contact, immediately wash with water and soap, and rinse thoroughly.
  - After eye contact, rinse opened eye for fifteen minutes under running water, then consult a doctor.
  - After swallowing, immediately call a doctor.
- A blood collection set must be used with the PAXgene Blood RNA Tube. See *Ordering Information*.**
- Practice Universal Precautions. Use gloves, gowns, eye protection, other personal protective equipment, and engineering controls to protect from blood splatter, blood leakage, and potential exposure to bloodborne pathogens.
- Handle all biologic samples and blood collection devices according to the policies and procedures of your facility. Obtain appropriate medical attention in the event of any exposure to biologic samples (for example, through a needle stick injury), since they may transmit viral hepatitis, HIV (AIDS), or other infectious diseases. Utilize any built-in needle protector, if the blood collection set provides one. PreAnalytiX does not recommend resheilding used needles. However, the policies and procedures of your facility may differ and must always be followed.
- Discard all blood collection tubes in biohazard containers approved for their disposal.
- Do not re-use the PAXgene Blood RNA Tubes.
- Do not use the PAXgene Blood RNA Tubes after the expiration date printed on the tube label.
- Since the PAXgene Blood RNA Tube contains a chemical additive, a blood collection set must be used for blood collection to prevent possible backflow from the tube.  
*See Specimen Collection and Preparation for Analysis section.*
- Excessive centrifugation speed (over 10,000 RCF) may cause PAXgene Blood RNA Tube breakage, exposure to blood and possible injury.
- Do not transfer a sample from a syringe to the PAXgene Blood RNA Tube.

## VII. Storage

Store the unused PAXgene Blood RNA Tubes at 18–25°C. Limited excursion temperatures up to 40°C are permitted.

## VIII. Instructions for Removal of BD Hemogard™ Closure

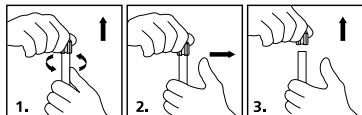- Grasp the PAXgene Blood RNA Tube with one hand, placing the thumb under the BD Hemogard Closure. (For added stability, place arm on solid surface). With the other hand, twist the BD Hemogard Closure while simultaneously pushing up with the thumb of the other hand **ONLY UNTIL THE TUBE STOPPER IS LOOSENED**.
- Move thumb away before lifting closure. **DO NOT** use thumb to push closure off tube. Caution: If the tube contains blood, an exposure hazard exists. To help prevent injury during closure removal, it is important that the thumb used to push upward on the closure is removed from contact with the tube as soon as the BD Hemogard Closure is loosened.
- Lift closure off tube. In the unlikely event of the plastic shield separating from the rubber stopper, **DO NOT REASSEMBLE CLOSURE**. Carefully remove rubber stopper from tube.

## IX. Technical Assistance

If you have any questions regarding the PAXgene Blood RNA System, contact one of the BD Technical Services Departments listed on BD – Customer Service section.

## X. Ordering Information

### PAXgene Products

#### PAXgene Blood RNA Tube / 762165

Content: 100 Blood collection tubes. Use with the PAXgene Blood RNA Kit (50)

2.5 ml • 16 × 100 mm	6.9 ml additive	Sterile
Plus • BD Hemogard	100 tubes/case	IVD

#### PAXgene Blood RNA Kit (50) 762164 N.A. / 762174 Ex. N.A.

Content: 50 PAXgene Spin Columns, Processing Tubes, RNase-free Reagents and Buffers.

Use with the PAXgene Blood RNA Tubes (100)

### BD Products\*

#### BD Vacutainer Push Button Blood Collection Set / 367344

Content: 21G 3/4 inch (0.8 × 19 mm) needle, 12 inch (305 mm) tubing with luer adapter. 50/box, 200/case

#### BD Vacutainer Safety-Lok Blood Collection Set / 367281 US / 367286 CE

Content: 21G 3/4 inch (0.8 × 19 mm) needle, 12 inch (305 mm) tubing with luer adapter. 50/box, 200/case

#### BD Vacutainer One Use Holder / 364815

Content: Case only for 13 mm and 16 mm diameter. 1000/case

#### BD Vacutainer Plus Serum Tubes / 367812 US / 368975 CE

Content: 13 × 75 mm 4.0 ml draw with red BD Hemogard closure and paper label. 100/box, 1000/case

\* These catalog numbers represent typical products that can be used with the PAXgene Blood RNA Tube.

### To order BD Blood Collection Accessories:

Go to: <http://www.bd.com/support/contact/international.asp>

## BD - Customer Service

### Argentina, Uruguay and Paraguay

Orders: 0800.444.5523  
E-mail: [crc\\_argentina@bd.com](mailto:crc_argentina@bd.com)

### Australia

Orders: 1.800.656.100  
Fax: 1.800.656.110  
E-mail: [bd\\_anz@bd.com](mailto:bd_anz@bd.com)

### Austria

Orders: 43.1.706.36.60  
Fax: 43.1.706.36.60-11  
Email: [customerservice.vac.ga@europe.bd.com](mailto:customerservice.vac.ga@europe.bd.com)

### Belgium

Orders: 32.53.720.556  
Fax: 32.53.720.549  
Orders: [orders.be@europe.bd.com](mailto:orders.be@europe.bd.com)

### Brazil

Orders: 0800.055.56.54  
E-mail: [consultoria\\_vacutainer@bd.com](mailto:consultoria_vacutainer@bd.com)

### Canada

Orders: 800.268.5430  
Fax: 800.565.0897  
E-mail: [custsvccan@bd.com](mailto:custsvccan@bd.com)

### Central and Eastern Europe (CEE)

Orders: 48.22.377.11.11  
Fax: 48.22.377.11.02  
Bulgaria: [info\\_bulgaria@europe.bd.com](mailto:info_bulgaria@europe.bd.com)  
Czech Republic: [info\\_czech@europe.bd.com](mailto:info_czech@europe.bd.com)  
Croatia: [info\\_croatia@europe.bd.com](mailto:info_croatia@europe.bd.com)  
Hungary: [info\\_hungary@europe.bd.com](mailto:info_hungary@europe.bd.com)  
Poland: [info\\_poland@europe.bd.com](mailto:info_poland@europe.bd.com)  
Romania: [info\\_romania@europe.bd.com](mailto:info_romania@europe.bd.com)  
SEE: [info\\_balkan@europe.bd.com](mailto:info_balkan@europe.bd.com)  
Serbia: [info\\_serbia@europe.bd.com](mailto:info_serbia@europe.bd.com)  
Slovakia: [info\\_slovakia@europe.bd.com](mailto:info_slovakia@europe.bd.com)  
Slovenia: [info\\_slovenia@europe.bd.com](mailto:info_slovenia@europe.bd.com)  
Technical: [bd\\_service\\_poland@bd.com](mailto:bd_service_poland@bd.com)

### Denmark

Orders: 45.43.43.45.66  
Fax: 45.43.96.56.76  
Orders: [ordire.dk@europe.bd.com](mailto:ordire.dk@europe.bd.com)  
Technical: [bddenmark@europe.bd.com](mailto:bddenmark@europe.bd.com)

### Finland

Orders: 358.9.88.70.780  
Fax: 358.9.88.70.7816  
Orders: [tilaukset.fi@europe.bd.com](mailto:tilaukset.fi@europe.bd.com)  
Technical: [bdsuomi@europe.bd.com](mailto:bdsuomi@europe.bd.com)

### France

Orders: 33.476.68.94.96  
Fax: 33.476.68.36.93  
Orders: [commandesfr@europe.bd.com](mailto:commandesfr@europe.bd.com)  
E-mail: [serviceclientbdf@europe.bd.com](mailto:serviceclientbdf@europe.bd.com)  
Technical: [vacutainerfr@bd.com](mailto:vacutainerfr@bd.com)

### Germany

Orders: 49.6221.305.553  
Fax: 49.6221.305.379  
Email: [customerservice.vac.ga@europe.bd.com](mailto:customerservice.vac.ga@europe.bd.com)

### Italy

Orders: 39.02.48.240.500  
Technical: 39.335.424.388  
Fax: 39.02.40.91.80.11  
E-mail: [ordini.it@europe.bd.com](mailto:ordini.it@europe.bd.com)

### Middle East & Africa (EMA)

Orders: 971.4.3379525  
Fax: 971.4.03379551  
E-mail: [EMA\\_PAS@bd.com](mailto:EMA_PAS@bd.com)

### The Netherlands

Orders: 31.20.582.94.20  
Fax: 31.20.582.94.21  
Orders: [orders.nl@europe.bd.com](mailto:orders.nl@europe.bd.com)

### New Zealand

Orders: 0800.572.468  
Fax: 0800.572.469  
E-mail: [nz\\_customerservice@bd.com](mailto:nz_customerservice@bd.com)

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Orders: 47.23.32.30.00  
Fax: 47.23.32.30.99  
E-mail: [kundeservice@puls-norge.no](mailto:kundeservice@puls-norge.no)

### Spain, Portugal and Andorra

Orders: 34.91.848.8104  
Technical: 34.902.27.17.27  
Fax: 34.91.828.81.48  
E-mail: [info.spain@europe.bd.com](mailto:info.spain@europe.bd.com)

### Sweden

Orders: 46.8.775.51.60  
Fax: 46.8.775.51.95  
Orders: [order.se@europe.bd.com](mailto:order.se@europe.bd.com)  
Technical: [bds sweden@europe.bd.com](mailto:bds sweden@europe.bd.com)

### Switzerland

Orders: 41.61.485.22.22  
Fax: 41.61.485.22.00  
E-mail: [infoch@europe.bd.com](mailto:infoch@europe.bd.com)

### UK

Orders: 44.1.865.781.666  
Fax: 44.1.865.717.313  
E-mail: [bduk\\_customerservice@europe.bd.com](mailto:bduk_customerservice@europe.bd.com)

### USA

Orders: 888.237.2762  
Technical: 800.631.0174  
Fax: 800.847.2220  
Technical: [www.bd.com/vacutainer/contact](http://www.bd.com/vacutainer/contact)

## QIAGEN – Customer Service

### Australia

Orders: 1.800.243.800  
 Technical: 1.800.243.066  
 Fax: 03.9840.9888

### Austria

Orders: 0800.28.10.10  
 Technical: 0800.28.10.11  
 Fax: 0800.28.10.19

### Belgium

Orders: 0800.79612  
 Technical: 0800.79556  
 Fax: 0800.79611

### Brazil

Orders: 0800.557779  
 Technical: 0800.557779  
 Fax: 55.11.5079.4001

### Canada

Orders: 800.572.9613  
 Technical: 800.DNA.PREP (800.362.7737)  
 Fax: 800.713.5951

### China

Orders: 86.21.3865.3865  
 Technical: 800.988.0325  
 Fax: 86.21.3865.3965

### Denmark

Orders: 80.885945  
 Technical: 80.885942  
 Fax: 80.885944

### Finland

Orders: 0800.914416  
 Technical: 0800.914413  
 Fax: 0800.914415

### France

Orders: 01.60.920.926  
 Technical: 01.60.920.930  
 Fax: 01.60.920.925

### Germany

Orders: 02103.29.12000  
 Technical: 02103.29.12400  
 Fax: 02103.29.22000

### Hong Kong

Orders: 800.933.965  
 Technical: 800.930.425  
 Fax: 800.930.439

### Ireland

Orders: 1800.555.049  
 Technical: 1800.555.061  
 Fax: 1800.555.048

### Italy

Orders: 800.789.544  
 Technical: 800.787.980  
 Fax: 02.334304.826

### Japan

Orders: 03.6890.7300  
 Technical: 03.6890.7300  
 Fax: 03.5547.0818

### Korea (South)

Orders: 080.000.7146  
 Technical: 080.000.7145  
 Fax: 02.2626.5703

### Luxembourg

Orders: 8002.2076  
 Technical: 8002.2067  
 Fax: 8002.2073

### Mexico

Orders: 01.800.7742.639  
 Technical: 01.800.7742.436  
 Fax: 01.800.1122.330

### The Netherlands

Orders: 0800.0229592  
 Technical: 0800.0229602  
 Fax: 0800.0229593

### Norway

Orders: 800.18859  
 Technical: 800.18712  
 Fax: 800.18817

### Singapore

Orders: 1800.742.4362  
 Technical: 1800.742.4368  
 Fax: 65.6854.8184

### Spain

Orders: 91.630.7050  
 Technical: 91.630.7050  
 Fax: 91.630.5145

### Sweden

Orders: 020.790282  
 Technical: 020.798328  
 Fax: 020.790582

### Switzerland

Orders: 055.254.22.11  
 Technical: 055.254.22.12  
 Fax: 055.254.22.13

### UK

Orders: 01293.422.911  
 Technical: 01293.422.999  
 Fax: 01293.422.922

### USA

Orders: 800.426.8157  
 Technical: 800.DNA.PREP (800.362.7737)  
 Fax: 800.718.2056

The PAXgene Blood RNA Tube is made in the UK by BD for PreAnalytiX GmbH.

PAXgene and PreAnalytiX are trademarks of PreAnalytiX GmbH; QIAGEN is a trademark of QIAGEN NV; Styrofoam is a trademark of The Dow Chemical Company; all other trademarks are property of Becton, Dickinson and Company.

 PreAnalytiX GmbH, Feldbachstrasse, 8634 Hombrechtikon, CH ©2015 PreAnalytiX GmbH [www.PreAnalytiX.com](http://www.PreAnalytiX.com)

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